

PROTOCOLLI DI APPLICAZIONE

APPLICATION PROTOCOL



GINOCCHIO

KNEE

ANCA

HIP



MANO

HAND

SPALLA

SHOULDER



CAVIGLIA

ANKLE

PRESENZA DI DOLORE

PRESENCE OF PAIN

GINOCCHIO/KNEE



T 0 gg 0 days	Regenflex STARTER AGO 21 G NEEDLE 21 G
T 8 gg 8 days	Regenflex BIO-PLUS AGO 19 G NEEDLE 19 G
T 6 mesi 12 months	

ANCA/HIP



T 0 gg 0 days	Regenflex STARTER AGO 20 G NEEDLE 20 G
T 8 gg 8 days	Regenflex BIO-PLUS AGO 20 G NEEDLE 20 G
T 6 mesi 6 months	

MANO/HAND



T 0 gg 0 days	Regenflex STARTER AGO 25 G NEEDLE 25 G
T 1 mese 1 month	
T 2 mesi 2 months	

SPALLA/SHOULDER



T 0 0 days	Regenflex STARTER AGO 21 G NEEDLE 21 G
T 8 gg 8 days	Regenflex BIO-PLUS AGO 19 G NEEDLE 19 G
T 6 mesi 6 months	

CAVIGLIA/ANKLE



T 0 gg 0 days	Regenflex STARTER AGO 21 G NEEDLE 21 G
T 8 gg 8 days	Regenflex BIO-PLUS AGO 19 G NEEDLE 19 G
T 6 mesi 6 months	

INDUTTIVO RIPARATIVO ANCHE CON DOLORE

INDUCTION CURE ALSO WITH PAIN

GINOCCHIO/KNEE



Regenflex **STARTER**
AGO 21 G
NEEDLE 21 G

Regenflex **BIO-PLUS**
AGO 19 G/NEEDLE 19 G


ANCA/HIP



Regenflex **STARTER**
AGO 20 G/NEEDLE 20 G

Regenflex **BIO-PLUS**
AGO 20 G/NEEDLE 20 G

MANO/HAND

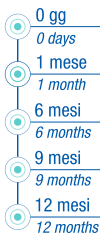

Regenflex **STARTER**
AGO 25 G
NEEDLE 25 G

SPALLA/SHOULDER



Regenflex **STARTER**
AGO 21 G
NEEDLE 21 G

Regenflex **BIO-PLUS**
AGO 19 G/NEEDLE 19 G

CAVIGLIA/ANKLE



Regenflex **STARTER**
AGO 21 G
NEEDLE 21 G

Regenflex **BIO-PLUS**
AGO 19 G/NEEDLE 19 G

TENDINITE e TENDINOSI

TENDONITIS & TENDINOSIS

GINOCCHIO / KNEE



ANCA / HIP



MANO / HAND



SPALLA / SHOULDER



CAVIGLIA / ANKLE



0 gg
0 days

Regenflex
STARTER
AGO 21 G
AGO 25 G
NEEDLE 21 G
NEEDLE 25 G

15 gg
15 days

Regenflex
STARTER
AGO 21 G
AGO 25 G
NEEDLE 21 G
NEEDLE 25 G

30 gg
30 days

Regenflex
STARTER
AGO 21 G
AGO 25 G
NEEDLE 21 G
NEEDLE 25 G

L'infiltrazione intra-articolare dell'anca va eseguita sempre ecoguidata.
The intra-articular infiltrations in the hip must be always ultrasound guided.